



Email : [sales@samcomedical.com](mailto:sales@samcomedical.com)  
Phone: 1-300-400-8211

**Product Name :**  
Samcomedical Operation Theater Fumigator

**Product Code :**  
SAMCOMEDICAL340001A001

**Description :**

Samcomedical Operation Theater Fumigator

**Technical Specification :**

Operation Theater Fumigator Widely demanded by several hospitals and nursing homes, our Operation Theater Fumigator are available in different models and sizes. Our fumigators consume negligible power and virtually require no maintenance that make these most demanded. Furthermore, our team of expert quality controllers thoroughly check all the products on various parameters to ensure flawlessness. Technical Specifications: Body: Fully Stainless Steel Maximum Height: 16 Inches Outer Diameter: 17 Inches Weight: 9 Kgs (Appr.) Electric Supply: 230 50Hz Single Phase A.C. Connected Load: 90 Watts Power Consumption: 0.150Kws / Hr. Atomizing Cap.: 30 - 35 ml / min Features: Light weight and portable Easy to operate virtually no maintenance required Negligible power consumption 100% convenient Disinfection Procedure: Unpack the equipment and check the functioning by plugging it into a pin 5 Amps 230 V electrical socket Ascertain the dosage required for the room to be disinfected Place 'OT Fumigator' inside the operation theater Pour in the formation & water into the dish from sides Close and seal all the windows, ventilators openings and the door of the place to be disinfected Plug in the unit's main cord into a 3 pin, 5 Amp, 230V, electrical sockets with switch. Which must be provided outside the operation theater or use automatic timer Switch on the 'OT Fumigator' and allow it to run for 30 minutes then switch it off Wait for 6 hours and open the room before re-using the room, exhaust remaining fumes if any by switching on the fan / exhaust of the window and air conditioner Operation Theater Fumigator, Operation Theater Fumigator Exporters, Operation Theater Fumigator Manufacturers



**Samco Medical**

Website: [www.samcomedical.com](http://www.samcomedical.com),