



Email : [sales@samcomedical.com](mailto:sales@samcomedical.com)  
Phone: 1-300-400-8211

**Product Name :**  
Hot Water Bottle Both Side Ribbed

**Product Code :**  
SAMCOMEDICAL500001A004



**Description :**

Hot Water Bottle Both Side Ribbed  
½ Unscrew the stopper ½ Fill warm water up to two third of the capacity  
½ Expel the air by pressing the bottle gently ½ Put the stopper and tight i ½ Bottle is ready for use  
Precautions: ½ Do not use boiling water ½ Keep one third of the bottle empty ½ After filling press gently to  
expel air ½ Stopper should not be loose ½ When not in use keep the empty bottle upside down after  
removing the stopper ½ Avoid mishandling by children Hot Water Bottle Both Side Ribbed, Hospital Rubbers  
Goods exporters, Hospital Rubbers Goods manufacturers



**Samco Medical**

Website: [www.samcomedical.com](http://www.samcomedical.com),